

Management of Skin Rejuvenation using JetPeel by TavTech

Prepared by Laura Masini, Clinical and Global Trainer for JetPeel, TavTech

Case Description

Patient Profile: Female, Age 60.

This case demonstrates the clinical efficacy of JetPeel by TavTech in treating advanced skin aging, including dermal laxity, deep wrinkles and impaired hydration, over two sessions. The subject, a 60 year old female showed reduced elasticity, prominent wrinkles and textural irregularities.

During the consultation, the following concerns were presented:

- **Uneven Tone and Texture:** Mild dyschromia and rough texture from photodamage and reduced collagen, affecting radiance and smoothness.
- **Wrinkles:** Fine dynamic forehead lines from muscle activity and collagen loss, disrupting a youthful appearance.
- **Laxity:** Subtle skin laxity, softening contours and reducing upper-face firmness.

Treatment Process

This protocol addressed all the concerns, including mild dyschromia, dynamic wrinkles and early skin laxity, over two sessions within 10 days. Treatments focused on refining texture, enhancing hydration, boosting collagen, and providing antioxidant protection, with 5-day intervals for optimal results without downtime.

TREATMENT 01	Wrinkle Reduction & Hydration	TREATMENT 02	Texture Refinement & Collagen Invigoration
	<ul style="list-style-type: none"> Glycolic acid 10% for gentle exfoliation and enhanced absorption. Argireline® peptide to relax dynamic wrinkles. Hyaluronic acid for deep hydration and improved elasticity. 		<ul style="list-style-type: none"> Mandelic & salicylic acids for exfoliation and pore refinement. Peptides to boost collagen and firmness. Vitamins A + E for antioxidant protection and cellular repair.

JetCare by TavTech products used in the JetPeel treatments

	LYMPHATIC MASSAGE STEP 01	EXFOLIATION STEP 02	INFUSION STEP 03	BOOSTER STEP 04
TREATMENT 1	18ml JetCare Hydro	5ml Renewal Care Glycolic Acid 10%	5ml Selective Care Argireline® Complex	3ml Boost Hydrate
TREATMENT 2	18ml JetCare Hydro	5ml Renewal Care Mandelic & Salicylic	5ml Anti-Aging Peptide Complex	3ml Boost Regenerate

T0: BEFORE JETPEEL TREATMENTS



T2: AFTER 2 JETPEEL TREATMENT SESSIONS



Summary and Conclusion

The treatment protocol effectively addressed the concerns, achieving visible improvements in texture, wrinkles, hydration, elasticity, pigmentation, and firmness after two sessions over 10 days. The skin appeared smoother, firmer and more radiant. For optimal results, an extended cycle of four additional sessions is recommended: two sessions spaced 5 days apart to enhance collagen and refine texture, followed by two weekly sessions to complete tone correction and hydration. A maintenance plan of one treatment every 3-4 weeks is advised to sustain hydration, collagen support, and long-term skin clarity and radiance.